




+ Dalbo® Abutment.

From the original to the system.

Original System by Cendres+Métaux.

Cendres+Métaux is constantly expanding its portfolio for overdenture systems on implants. With the introduction of the Dalbo[®] Abutment, Cendres+Métaux enhance a direct and complete access to the famous and clinically outstanding Dalbo[®]-System. The functioning principle has been in use for over 40 years and has clinically proven its effectiveness in millions of cases.

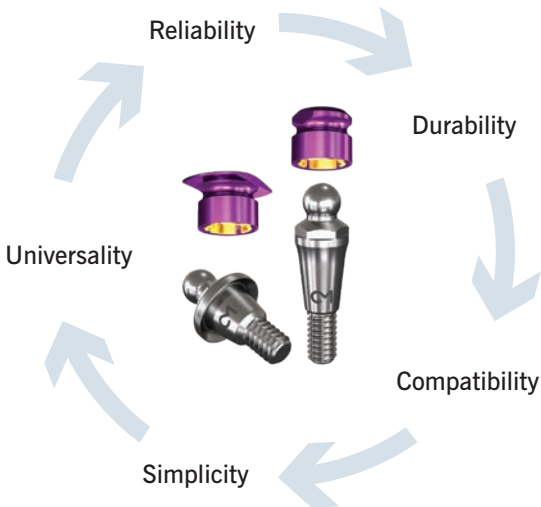

Your advantages.

Reliability and durability

The compact spherical shape is highly resistant. The special design and the combination with precious metal inserts in Elitor® ensure reliable and durable functioning.

Simplicity

The Dalbo®-System is easy to use for the practice and laboratory and requires minimum servicing.

Universal use

The system components have been optimized for the relevant application.

Compatibility

All female parts fit the spherical male parts of other manufactures of spherical ball attachments with a diameter of 2.25 mm.


Adjusting the retention strength.

The Dalbo®-PLUS offers the unique feature that the retention can easily be adjusted according to the patient's mouth situation and requirements. The retention strength increases by approx. 200g with each $\frac{1}{4}$ rotation.

If needed, the lamellae retention insert can be exchanged, or in case of advanced wear of the sphere, replaced with a tuning lamellae retention insert without removing the female part from the denture's body.




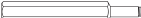


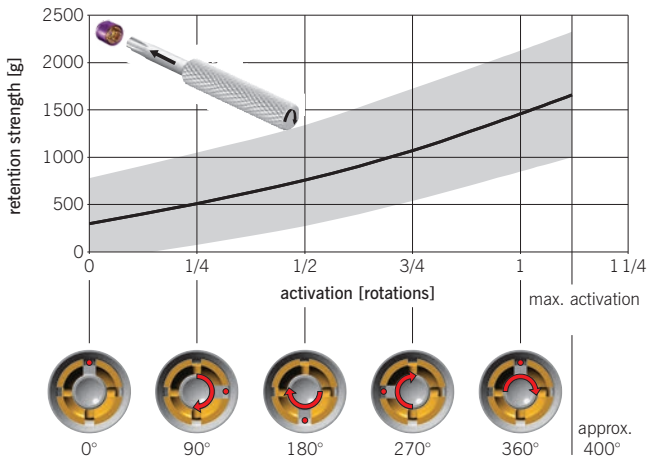

The highest level of activation is obtained after one and a half clockwise rotations (max. 2 200g).

Retention strength in relation to activation



Portfolio.

Order No.		Description	Contents
see web		Dalbo® Abutment	1 pc. NEW
05002599		Dalbo® CAD/CAM Retention Element Retention Element for Bar	1 pc. NEW
055752		Female part TE basic complete	1 pc.
055890		Female part TE elliptic complete	1 pc.
055643		Lamellae retention insert E	1 pc.
0500068		Tuning lamellae retention insert soft E	1 pc.
055687		Tuning lamellae retention insert E	1 pcs.
050394		Spacer disc Z	5 pc.
072626		Duplicating aid / spacer G	1 pc.
070157		Transfer jig	1 pc.
07000312		Dalbo® Abutment Analog	4 pcs. NEW

Order No.	Description	Contents
0700 0266	Dalbo® Abutment Screw Driver	1 pc.
		
072 609	Screwdriver/activator	1 pc.
		
072 639	Heating rod	1 pc.
		
0700 0026	Gauge set	1 pc.
	<p data-bbox="306 719 391 739">Includes:</p> <p data-bbox="306 745 594 765">Male part gauge (07000027)</p> <p data-bbox="306 771 615 792">Female part gauge (07000024)</p> <p data-bbox="306 797 646 818">Lamellae retention insert (055643)</p> <p data-bbox="306 824 778 844">Tuning lamellae retention insert soft (05000068)</p> <p data-bbox="306 850 708 870">Tuning lamellae retention insert (055687)</p> <p data-bbox="306 876 605 896">Screwdriver/activator (072 609)</p>	
0700 0269	Torque Wrench incl. Driver Handle	1 pc.
		
0700 0268	Torque Wrench Driver Handle	1 pc.
		

Availability.

Dalbo® Abutment solutions are available on all major implant systems like:

- Straumann®
- Astra Tech®
- Nobel Biocare®
- DENTSPLY Ankylos®
- Camlog®
- Osstem®



Compatibility list available online:
www.cmsa.ch/Dental/Download-Center

Products marked with ® are registered trademarks of the respective manufacturer.